Die neue Generation.

BINZ
AMBULANCE

Ambulance 2000

BINZ

AMBULANCE

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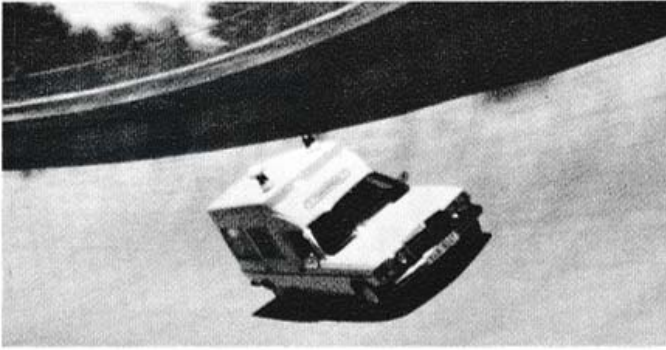
BINZ-AMBULANCE.
 This concept stands for experience and know-how of a well-known ambulance manufacturer. Thorough development work and product planning with the goal to bring help to sick and injured people in a fast, gentle and effective manner are the steps of the BINZ-AMBULANCE success. A complete line of ambulance cars, emergency ambulance vans and physician's emergency ambulance vans are manufactured in modern, spacious manufacturing plants. World-wide exchange of experience in the complete field of the emergency rescue technique and frequent contact with the practical daily work are the rudiments for BINZ-AMBULANCE cars which help people – with modern technique.

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2/3/4/5/6 Before the new BINZ AMBULANCE 2000 was put out into production it was thoroughly tested for service life, behaviour at high speed, on sharp corners, on rough surfaces and in wet and icy conditions. Exact measurements were recorded for the vehicle body and dummy driver and patient. Final alterations were then made. As a result, optimum harmony between chassis and body is guaranteed, which means high safety and comfort.

7 The body is the basis. Pressed parts fabricated in our plant are welded together with the help of gauges.

8 Stove enamel painting of the car bodies in paint-boxes with the following hardening process of high-elastic resin paint layers in tunnel drying chambers guarantee long life and protection against corrosion. After that the car becomes an ambulance. At the assembly line all special equipments for the emergency rescue technique are built in by electricians, saddlers, carpenters and fitters.

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9/10 A new concept: wider and higher passenger room with ideal dimensions. The result: better working space in the roomy ambulance compartment, more free height above the patient, a better infusion height, higher and wider rear door opening, more storage possibilities. The BINZ AMBULANCE development for optimal patient attendance and best transportation comfort. BINZ AMBULANCE 2000 long bodies are built on Mercedes-Benz chassis of the type 240 D, 300 D as well as 250 with extended wheelbase 3425, standard level regulation and hydrosteering.

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11 Sliding platform as main stretcher support. Fully extractable for easy loading and unloading of the stretcher. The one-part rear-wall flap-gas spring supported – opens to the top and thus offers protection against rain and snow. Additional warning lights at the top corners of the opened flap give additional security. The rear flash light which can be swivelled and revolved to all sides facilitates work in the night and can be additionally used for illuminating the scene of accident.

12 In addition to the normal position the sliding platform can be shifted up to the partition-wall in order to offer an optimum of comfort during long-distance transportation and for better attendance. The big roof storage space above the driver's cab is accessible from the patient's compartment – alternatively with or without flap. Additional illumination through neon tubes. Patient's and attendant's seat (back seat), adjustable, on the right side, facing direction of motion, with collapsible backrest.

13 The widely opening rear flap – lifting up beyond the rear edge of the roof – guarantees that the loading area is protected against rain etc. This loading location enables the attendant to reach the patient on the entire length of the stretcher.

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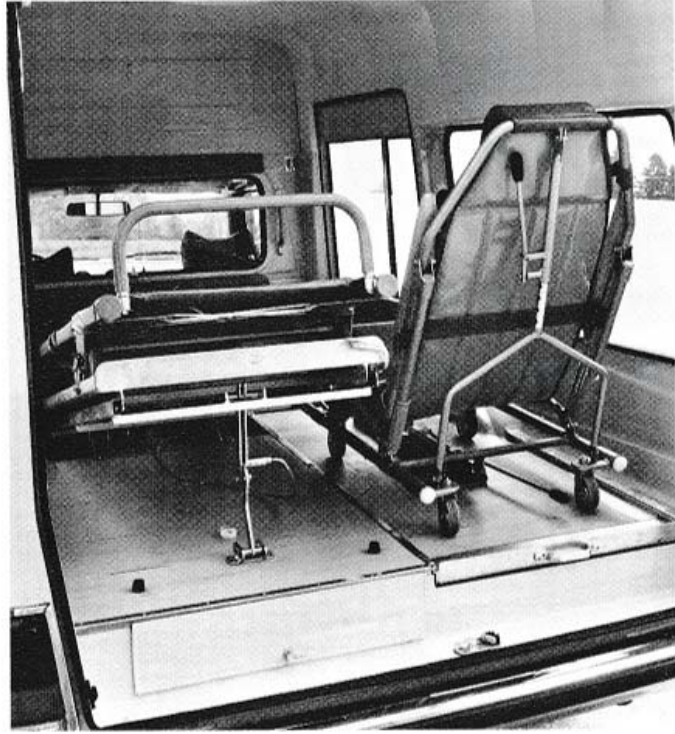
14 Optimum attendance facilities for the patient from the head part resp. and/or from the side. Automatic seat belt as well as head-rest for attendant's seat.

15 Device for lowering the head acc. to Trendelenburg. To be operated by the attendant by means of a foot pedal at the top end of the platform even during transportation.

16 Two-stage Trendelenburg position of lowering the head (9/13 degrees). A second stretcher may be used even when an installation fixture for a reclining chair is provided. In this case the left stretcher guiding rail is usually stored below the intermediate floor. All BINZ AMBULANCE 2000 models can also be equipped

with a second stretcher on the right side. It is loaded by means of an extractable platform.

The jointlessly PVC-covered intermediate floor is hygienic and can easily be cleaned.

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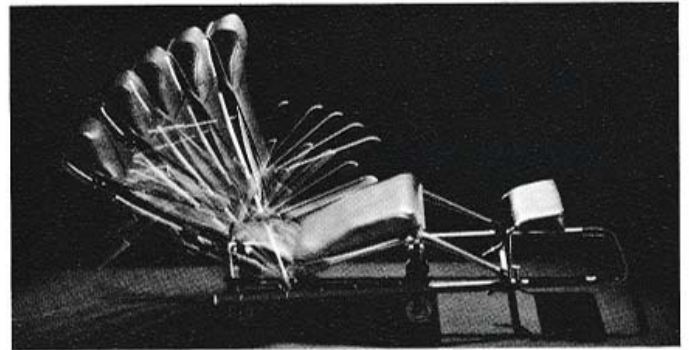
17/18 Reclining chair to be used as portable chair for transportation inside a building and as a chair for a sitting patient inside the car. With into numerous positions easily adjustable backrest up to lying position. Therefore also to be used as an emergency stretcher – with four wheels two of which are of swivelling type, built in head rest – belt – arm and foot rests – centre chair guiding. Possible to be fixed inside the car with face in front or rear. Easy to be folded.

19/20 Carrying chair without adjustable back part but fully foldable – with wheels. If desired with head rest. Especially to be used for transportation inside a building.

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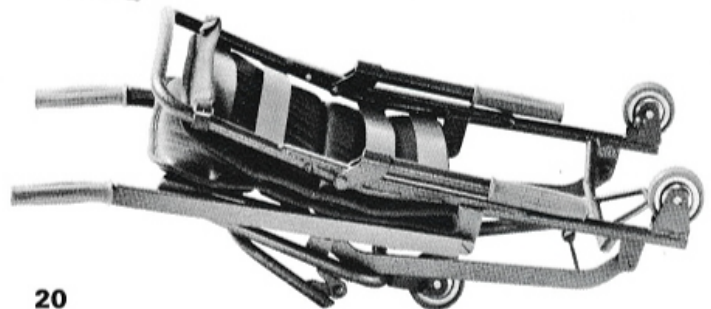
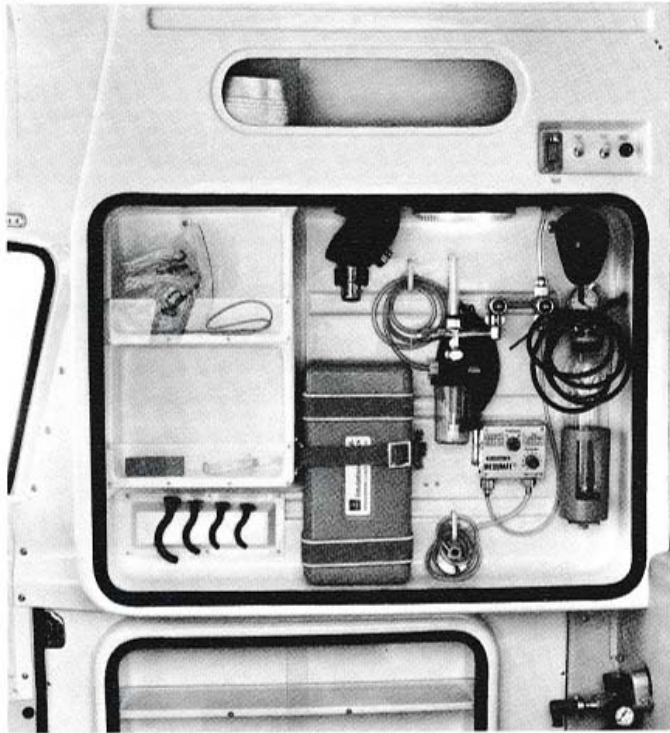
23

21/22/23 The reclining chair is shifted into the car via a platform – to be used in or counter direction of motion. In special cases it may be used as an emergency stretcher.

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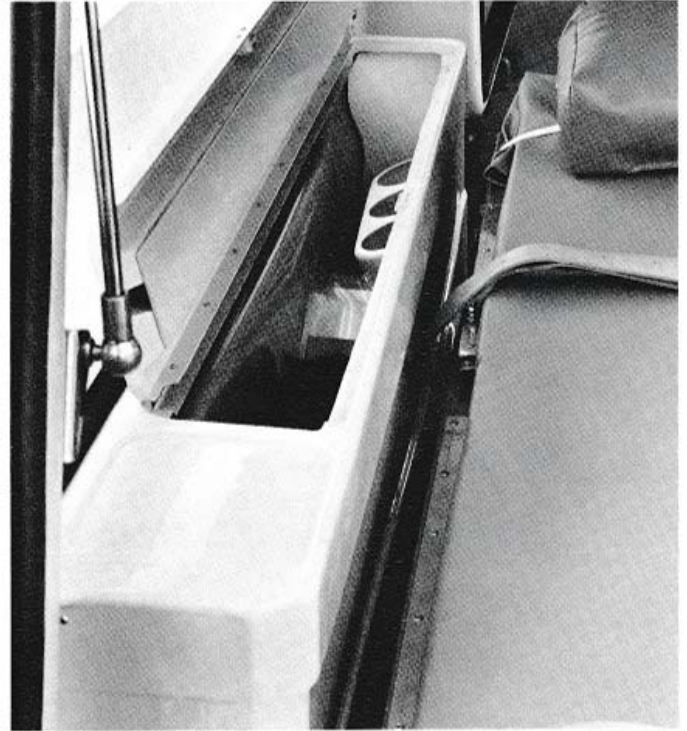
24 Suspended plastic cupboard at the left back door. With plexiglass sliding panes which permit easy access to the securely stored instruments such as oxygen and secretion-suction unit including accessories. Simple cleaning. Inside security guaranteed by sunk-mounted switches and rounded edges.

25 Solid but light plastic storage cabinet along the left side-wall. Accessible from the top by a lid. Especially for the storage of splints, infusions bottles etc.

26 The ABS-plastic cabinet mounted into the left rear door is clearly arranged and


25

26

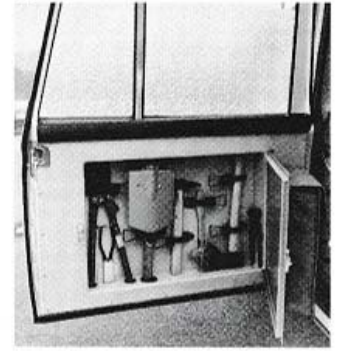27

easy accessible from the assistant's seat. Two plexiglass-sliding panes are covering the case for easy-accessible emergency devices.

27 As an alternative for dressing material storage case a tool box can be fitted into the left back door. The rescue tools are clearly arranged. A leatherette pouch for bed pan, kidney shaped bowl and urin bottle is fastened on the tool box lid.

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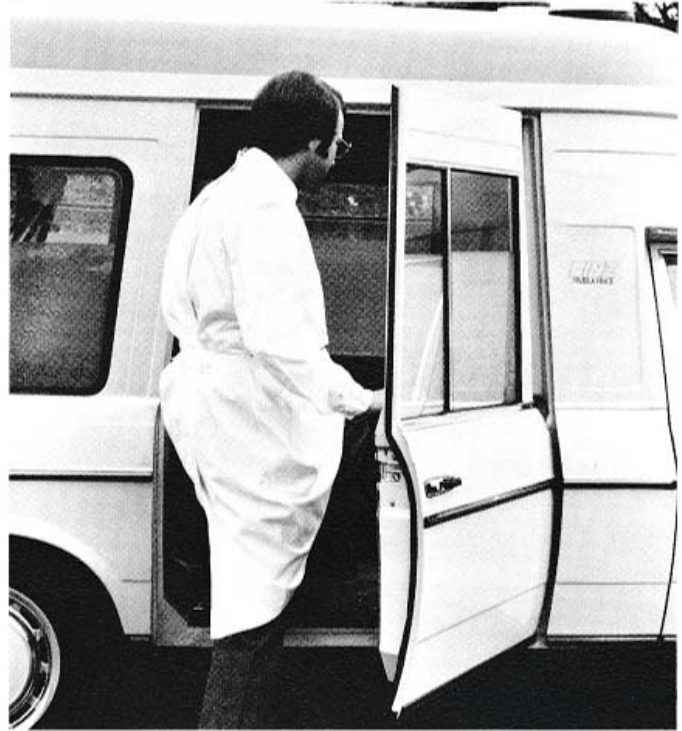
28/29 Instead of the standard flap below the intermediate floor a drawer may be installed on request. Accessible even if stretchers are placed.

30 The drawer below the intermediate floor can be filled with easy accessible rescue and vehicle tools as well.

31 Extra high rear side door with additionally extended height by 140 mm to an overall of 1240 mm free height. The entire width of the door is available for entrance without being narrowed by the wheel housing.

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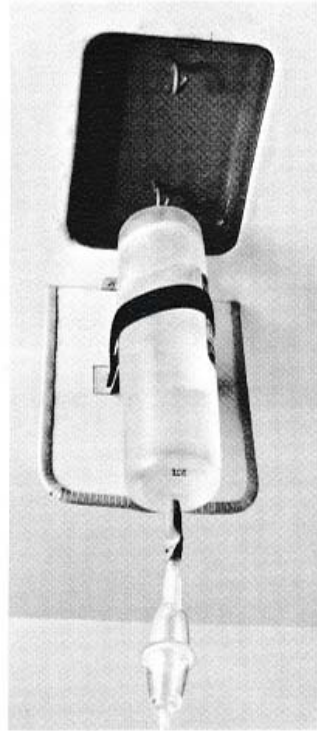
34

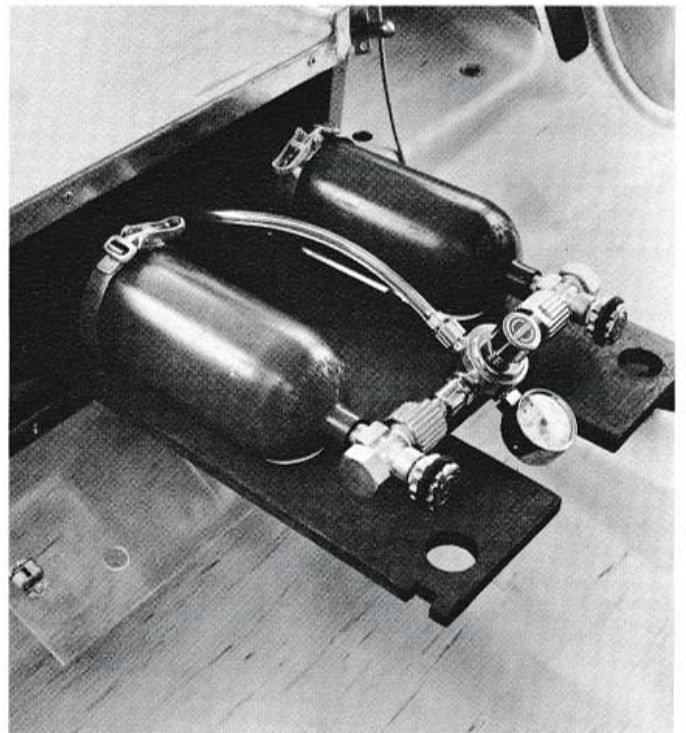
32/33 Special execution of the right back seat, may alternatively be used as patient's seat, facing direction of motion, or as attendant's seat (backrest = seat pad), facing the rear.

34 Suspension device for two infusion bottles with anti-swing fixture, integrated into the ceiling. Covered by a flap.

35 Secure storage of oxygen bottles (two units/5 l contents) below the intermediate floor, right-hand. Simple exchange by towing device.

Manometer control can be effected from attendant's seat. The storage guarantees a well-balanced weight and thus a more favourable centre of vehicle gravity.

36 Oxygen units model Dräger or others (inhaler with or without suction device for secretions, Pulmotor units etc.). On request portable version.

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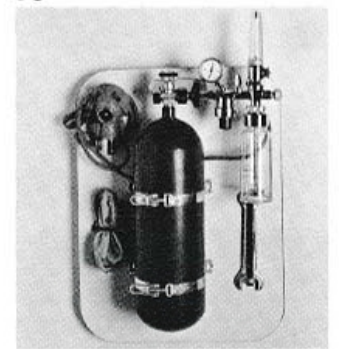
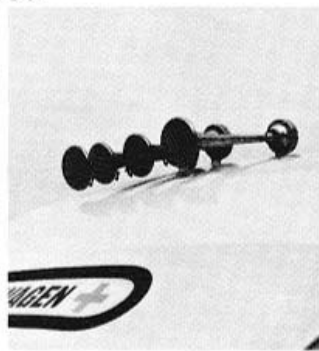
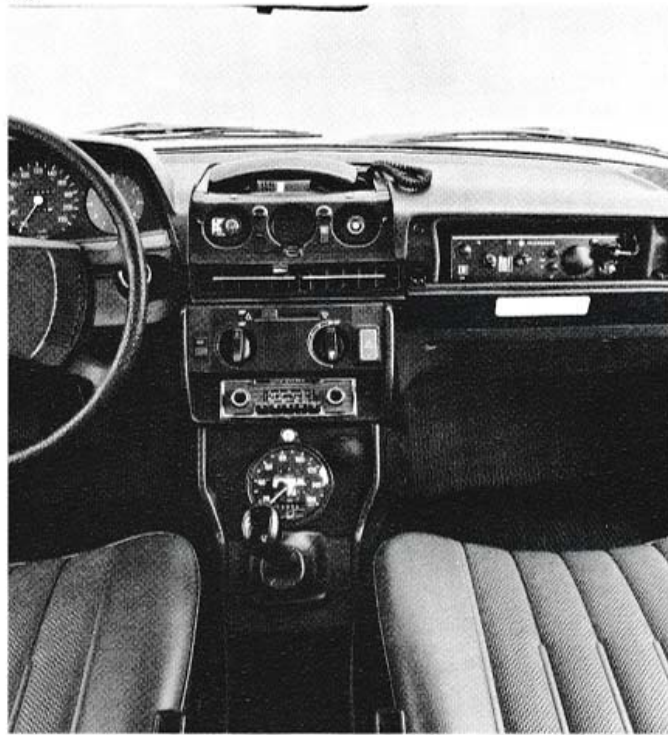
37/38 An illuminated transparent can be taken instead of a standard identification light at the front part of the roof.

39 Signal horn units, loudtone horns or compressor fanfares in various executions. Visual warning systems: big and small Halogen rotating beacons, one or two units, in symmetric or staggered arrangement.

40 More favourable sitting position for driver and co-driver acc. to ECE-regulations. Suitable instrumentation as well as easy operation of the wireless-set.

41 The spare wheel, if desired with cover, is stored behind the assistant driver's seat in standing position. Simple, essential rescue tooling, fire-extinguisher, warning flash lights etc., always ready on hand, stored in driver's cab.

42 Patient's and driver's compartment may be furnished with an air-conditioner which is dependant on the engine for pleasantly tempered room.

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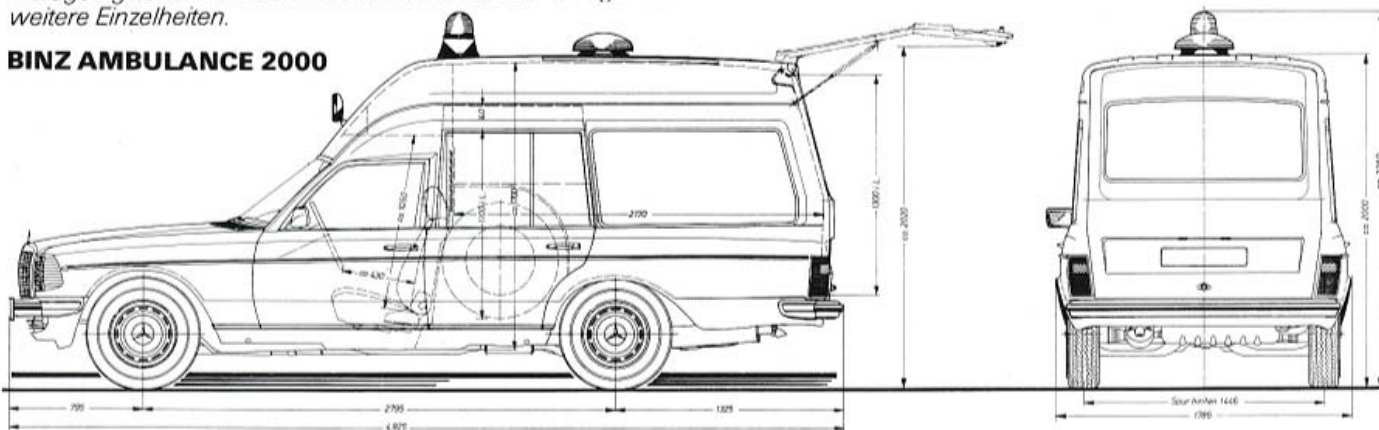
BINZ AMBULANCE 2000

Technical chassis data:

BINZ AMBULANCE 2000 bodies are also built on Mercedes-Benz chassis with standard wheelbase (2795 mm). Apart from two-stretcher-equipments there are many alternatives. Please ask for further information if required.

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BINZ GmbH & Co.